UK

An active bandage to support and treat bruising of the chest and ribcage. This bandage provides optimum fit and comfort to give the ribs support and protection.

#### **Applying the Bandage**

The Rib bandage is wrapped around the chest and can be fixed by adjustable Velcro straps. The pressure, compression of this bandage is adjustable and easy to adjust by means of the Velcro closure.

#### **Care and Cleaning**

Wash in warm water (by hand) with a mild detergent, rinse thoroughly and hang dry. Do not wash in the machine or dry clean. Do not tumble dry.

#### **WARNING**

Do not wear on open wounds or if you have a tendency to skin disorder, allergic reaction or circulatory disorder.

- The bandage is intended for the individual care of the patient.
- In case of improper use, product liability is excluded.
- A combination with other products must be discussed in advance with the doctor.
- If the symptoms increase or you notice abnormal changes, please consult your doctor.
- Please do not allow the bandage to come into contact with fatty and acidic agents, ointments and lotions.

**Not a substitute for medical care. Always consult a doctor for injury diagnosis and treatment.**

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