Its high quality and convenience of use makes this patella strap **ideal for use in sports**. It offers support when suffering from stabbing pain below the knee, by pressing onto the knee tendon with gel pads and supporting the patella (knee cap). This relieves strain at the tendon's attachment of the upper leg muscle and reduces pull on the lower leg, thus relieving the pain. Whereas other patella braces sometimes become loose or give insufficient support, this is not the case with the MED patella brace by Schaper. This patella strap is easy and fast to **close on two sides**. In addition, the patella brace can be washed without any problems.

#### Applying the Bandage

Place the patella band centered below the knee cap and close the Velcro fastener.

#### Care and Cleaning

Wash in warm water (by hand) with a mild detergent, rinse thoroughly and hang dry. Do not wash in the machine or dry clean. Do not tumble dry.

**WARNING** Do not wear on open wounds or if you have a tendency to skin disorder, allergic reaction or circulatory disorder.

- The bandage is intended for the individual care of the patient.
- In case of improper use, product liability is excluded.
- A combination with other products must be discussed in advance with the doctor.
- If the symptoms increase or you notice abnormal changes, please consult your doctor.
- Please do not allow the bandage to come into contact with fatty and acidic agents, ointments and lotions.

**Not a substitute for medical care. Always consult a doctor for injury diagnosis and treatment.**

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